



Cross reactivities on abc rapid drug tests

Medicine (Trade name)

Content / Active substance

Result when tested with abc

Acephen	Acetaminophen (see also Paracetamol)	No Cross reaction
Aceta	Acetaminophen (see also Paracetamol)	No Cross reaction
Actinac	Hydrocortisone	No Cross reaction
Actiq	Fentanyl	No Cross reaction
Adalat	Nifedipine	No Cross reaction
Adam (slang)	Methylenedioxyamphetamin (MDMA)	Positive on Methamphetamine (abc-MET) and Ecstasy (abc-MDMA)
Adderall	Dexamphetamine Sulphate	Positive on Amphetamine (abc-AMP)
Adderall XR	Dexamphetamine Sulphate	Positive für Amphetamine (abc-AMP)
AeroBec	Beclometasone	No Cross reaction
AeroBec Forte	Beclometasone	No Cross reaction
Aerocrom	Salbutamole	No Cross reaction
Aerolin	Salbutamole	No Cross reaction
Afrin	Oxymetazoline	No Cross reaction
Afrinol	Pseudoephedrine	No Cross reaction
Airomir	Salbutamole	No Cross reaction
Akineton	Biperiden	No Cross reaction
Aleve	Naproxen	No Cross reaction
Alfenta	Alfentanil	No Cross reaction
Alisone	Aluminium Hydroxyde	No Cross reaction
Allerfen	Promethazine	Positive on Tricyclic Antidepressants (abc-TCA)
Alphaderm	Hydrocortisone	No Cross reaction
Alphosyl-HC	Hydrocortisone	No Cross reaction
Alu-Cap	Aluminium Hydroxyde	No Cross reaction
Alurate	Aprobarbital	Positive on Barbiturater (abc-BAR)
Amoram	Amoxicilline oder Amoxycilline	No Cross reaction
Amoxil	Amoxicilline oder Amoxycilline	No Cross reaction
Amplan	Pericyazine	No Cross reaction
Amytal	Amobarbital	Positive on Barbiturates (abc-BAR)
Anabact	Metronidazole	No Cross reaction
Anadin	Acetylsalicylic acid, Aspirin, ASS	No Cross reaction
Anasin	Acetylsalicylic acid, Aspirin, ASS	No Cross reaction
Andropatch	Testosterone	No Cross reaction
Anergan	Promethazine	Positive on Tricyclic Antidepressants (abc-TCA)
Angettes	Acetylsalicylic acid, Aspirin	No Cross reaction
Angitak	Isosorbide Dinitrate	No Cross reaction
Anhydrol Forte	Aluminium Chloride Hexahydrate	No Cross reaction
Antaxone	Naltrexone	No Cross reaction
Antinaus	Promethazine	Positive on Tricyclic Antidepressants (abc-TCA)
Anugesic-HC	Hydrocortisone	No Cross reaction
Anusol-HC	Hydrocortisone	No Cross reaction
Anxitol	Medazepam	Positive on Benzodiazepine (abc-BZD)
Aolept	Pericyazine	No Cross reaction
Apacet	Acetaminophen (see also Paracetamol)	No Cross reaction
Apamin	Pericyazine	No Cross reaction
Aprinox	Bendrofluazide	No Cross reaction



Cross reactivities on abc rapid drug tests

Medicine (Trade name)

Content / Active substance

Result when tested with abc

Centrax	Prazepam	Positive on Benzodiazepine (abc-BZD)
Ceporex	Cephalexin	No Cross reaction
Chlorohex	Chlorhexidines Gluconate	No Cross reaction
Cidomycin	Gentamicin Sulphate	No Cross reaction
Cipramil	Citalopram	No Cross reaction
Claritin	Loratadine	No Cross reaction
Climagest	Norethindrone	No Cross reaction
Climesse	Norethindrone	No Cross reaction
Clonopin	Clonazepam	Positive on Benzodiazepine (abc-BZD)
Co-codamol	Paracetamol (Acetaminophen)/ Codeine Preparations	Positive on Opiates (abc-OPI)
Codafen	Ibuprofen	No Cross reaction
Codafen	Paracetamol (Acetaminophen)/ Codeine Preparations	Positive on Opiates (abc-OPI)
Codafen Continus	Codeine Phosphate	Positive on Opiates (abc-OPI)
Codeine Linctus	Codeine Phosphate	Positive on Opiates (abc-OPI)
Codidol ret.	Retardiertes Dihydrocodein	Positive on Opiates (abc-OPI)
Co-dydramol	Paracetamol (Acetaminophen)/ Codeine Preparations	Positive on Opiates (abc-OPI)
Co-fluampicil	Flucloxacillin	No Cross reaction
Cogaretic	Bendrofluzide	No Cross reaction
Colioam	Hydrocortisone	No Cross reaction
Combicillin	Penicillin	No Cross reaction
Combivent	Salbutamol	No Cross reaction
Compensan ret.	Retardiertes Morphine hydrochloride	Positive on Opiates (abc-OPI)
Condrotec	Naproxen	No Cross reaction
Convulex	Sodium Valproate	No Cross reaction
Co-proximal	Dextropropoxyphene	Positive für Propoxyphene (PPX)
Cordsodyl	Chlorhexidine Gluconate	No Cross reaction
Corgard	Nadolol	No Cross reaction
Corgaretic	Nadolol	No Cross reaction
Corgartic	Nadolol	No Cross reaction
Co-tenidone	Atenolol	No Cross reaction
Coumadin	Warfarin Sodium	No Cross reaction
Crystapen	Benzylpenicillin	No Cross reaction
Curral	Allobarbital	Positive on Barbiturates (abc-BAR)
CX Powder	Chlorhexidine Gluconate	No Cross reaction
Cyclimorph	Morphine	Positive on Opiates (abc-OPI)
Daktacort	Hydrocortisone	No Cross reaction
Dapacen	Acetaminophen (Siehe auch Paracetamol)	No Cross reaction
Darvocet	Dextropropoxyphene	Positive für Propoxyphene (PPX)
Darvon	Dextropropoxyphene	Positive für Propoxyphene (PPX)
Decadron	Dexamethasone	No Cross reaction
Delta-cortef	Prednisolone	No Cross reaction
Demerol	Meperidine	No Cross reaction
Demerol	Pethidine (meperidine)	No Cross reaction
Demetrin	Prazepam	Positive on Benzodiazepine (abc-BZD)
Depakene	Sodium Valproate	No Cross reaction
Depakote	Sodium Valproate	No Cross reaction



Cross reactivities on abc rapid drug tests

Medicine (Trade name)

Content / Active substance

Result when tested with abc

Medicine (Trade name)	Content / Active substance	Result when tested with abc
Depixol	Flupentixol decanoate	No Cross reaction
Dermol	Chlorhexidine Gluconate	No Cross reaction
Desoxyn	d-Methamphetamine HCL	Positive on Methamphetamine (abc-MET)
Desyrel	Trazodone	No Cross reaction
Dexa-Rhinaspray Duo	Dexamethasone	No Cross reaction
Dexedrine	Dexamphetamine Sulphate	Positive für Amphetamine (AMP)
DHC Continus	Dihydrocodeine	Positive on Opiates (abc-OPI)
Diazemuls	Diazepam	Positive on Benzodiazepine (abc-BZD)
Dicloflex	Diclofenac Sodium	No Cross reaction
Diclomax	Diclofenac Sodium	No Cross reaction
Difflam	Benzydamine HCl	No Cross reaction
Dilantin	Phenytoin	No Cross reaction
Dilaudid	Hydromorphone	Positive on Opiates (abc-OPI)
Dimetapp	Bromopheniramine	No Cross reaction
Dimotane	Bromopheniramine	No Cross reaction
Dimotapp	Bromopheniramine	No Cross reaction
Diocalm	Koalin und Morphine Mixture	Positive on Opiates (abc-OPI)
Dioderm	Hydrocortisone	No Cross reaction
Dionin	Ethylmorphine	Positive on Opiates (abc-OPI)
Diprozin	Promethazine	No Cross reaction
Disipil	Orphenadrine	No Cross reaction
Disprin	Acetylsalicylic acid, Aspirin, ASS	No Cross reaction
Dolantin	Pethidine (meperidine)	No Cross reaction
Dolophine	Methadone Hydrochloride	Positive on Methadone (abc-MTD)
Dolormin	Ibuprofen	No Cross reaction
Donnatal	Phenobarbital	Positive on Barbiturates (abc-BAR)
Drichlor	Aluminium Chloride Hexahydrate	No Cross reaction
Dropletan	Droperidol	No Cross reaction
Duovent	Salbutamol	No Cross reaction
Duphalec	Lactulose	No Cross reaction
Duramorph	Morphine	Positive on Opiates (abc-OPI)
Dyspamet	Cimetidine	No Cross reaction
E (slang)	Methylenedioxyamphetamine (MDMA)	Positive on Metamphetamine (abc-MET) and Ecstasy (abc-MDMA)
Econacort	Hydrocortisone	No Cross reaction
Econopred	Prednisolone	No Cross reaction
Ecotrin	Acetylsalicylic acid, Aspirin, ASS	No Cross reaction
Ecstasy	Methylenedioxyamphetamine (MDMA)	Positive on Metamphetamine (abc-MET) and Ecstasy (abc-MDMA)
Efcortesel	Hydrocortisone	No Cross reaction
Effexor	Venlafaxine	Potential Positive on abc- PCP
Effexor XL	Venlafaxine	Potential Positive on abc- PCP
Efortelan	Hydrocortisone	No Cross reaction
Elavil	Amitriptyline	Positive on Tricyclic Antidepressants (abc-TCA)
Elleste Duet	Norethindrone	No Cross reaction
Eltroxin	Thyroxin Sodium	No Cross reaction
Elyzol	Methronidazole	No Cross reaction
Empirin	Acetylsalicylic acid, Aspirin	No Cross reaction



Cross reactivities on abc rapid drug tests

Medicine (Trade name)

Content / Active substance

Result when tested with abc

Medicine (Trade name)	Content / Active substance	Result when tested with abc
Entersan	Koalin und Morphin Mixture	Positive on Opiates (abc-OPI)
Epanutin	Phenytoin	No Cross reaction
Epilim	Sodium Valproate	No Cross reaction
Epitard	Phenytoin	No Cross reaction
Ergenyl	Sodium Valproate	No Cross reaction
Erycane	Erythromycin	No Cross reaction
Erymax	Erythromycin	No Cross reaction
Erythrocin	Erythromycin	No Cross reaction
Erythroped	Erythromycin	No Cross reaction
Estracombi	Norethindrone	No Cross reaction
Eurax-HC	Hydrocortisone	No Cross reaction
Eve (slang)	Methylenedioxyamphetamine (MDA)	Positive on Amphetamine (abc-AMP) and Ecstasy (abc-MDMA)
Evorel	Norethindrone	No Cross reaction
Excedrin	Acetylsalicylic acid, Aspirin, ASS	No Cross reaction
Excedrin (combination)	Acetaminophen (siehe auch Paracetamol)	No Cross reaction
Excedrin (combination)	Acetylsalicylic acid, Aspirin, ASS	No Cross reaction
Fancorsil	Dimenthicone	No Cross reaction
Fenbid	Ibuprofen	No Cross reaction
Fenergan	Promethazine	Positive on Tricyclic Antidepressants (abc-TCA)
Feverall	Acetaminophen (siehe auch Paracetamol)	No Cross reaction
Fillair	Beclometasone	No Cross reaction
Finlepsin	Carbamazin	No Cross reaction
Fioricet	Butalbital	Positive on Barbiturates (abc-BAR)
Fiorinal	Butalbital	Positive on Barbiturates (abc-BAR)
Flagyl	Metronidazole	No Cross reaction
Flexin continus	Indomethacin	No Cross reaction
Floxapen	Flucloxacillin	No Cross reaction
Fluanxol	Flupentixol decanoate	No Cross reaction
Fortagesic	Paracetamol (Acetaminophen)	No Cross reaction
Frisium	Clobazam	Positive on Benzodiazepine (abc-BZD)
Fucibet	Fusidic Acid (Sodium Fusidate)	No Cross reaction
Fucidin	Fusidic Acid (Sodium Fusidate)	No Cross reaction
Fucithalmic	Fusidic Acid (Sodium Fusidate)	No Cross reaction
Galcodine	Codeine Phosphate	Positive on Opiates (abc-OPI)
Galenphol	Pholcodine	Positive on Opiates (abc-OPI)
Galpseud	Chlorpheniramine Maleate	No Cross reaction
Gamanil	Lofepamine	Positive on Tricyclic Antidepressants (abc-TCA)
Gastrocote	Aluminium Hydroxide	No Cross reaction
Gaviscon	Magnesium Alginate	No Cross reaction
Gaviscon	Mannitol	No Cross reaction
Genticin	Gentamicin Sulphate	No Cross reaction
Gentisone	Gentamicin Sulphate	No Cross reaction
Glianimon	Benperidol (Butyrophenone)	No Cross reaction
Gregoderm	Hydrocortisone	No Cross reaction
Halcion	Triazolam	Positive on Benzodiazepine (abc-BZD)
Haldol	Haloperidol	No Cross reaction



Cross reactivities on abc rapid drug tests

Medicine (Trade name)

Content / Active substance

Result when tested with abc

Largactil	Chlorpromazine	No Cross reaction
Lentizole	Amitriptyline	Positive on Tricyclic Antidepressants (abc-TCA)
Lerisum	Medazepam	Positive on Benzodiazepine (abc-BZD)
Levium	Levomepromazin	No Cross reaction
Lexotan	Bromazepam	Positive on Benzodiazepine (abc-BZD)
Librium	Chlordiazepoxide	Positive on Benzodiazepine (abc-BZD)
Lipitor	Atorvastatin	No Cross reaction
Locoid	Hydrocortisone-17-butyrate	No Cross reaction
Locoid C	Hydrocortisone-17-butyrate	No Cross reaction
Loestrin	Norethindrone	No Cross reaction
Lomont	Lofepamine	Positive on Tricyclic Antidepressants (abc-TCA)
Lomotil	Co-Phenotrope (atropine/diphenoxylate)	No Cross reaction
Loperagen	Loperamide HCl	No Cross reaction
Loratab	Hydrocodone	Positive on Opiates (abc-OPI)
Lorcet	Hydrocodone	Positive on Opiates (abc-OPI)
Love Drug (slang)	Methylenedioxyamphetamine (MDA)	Positive on Amphetamine(abc-AMP) and Ecstasy (abc-MDMA)
LSD	Lysergic acid diethylamide	No Cross reaction
Luminal	Phenobarbitone (sehen auch Phenobarbital)	Positive on Barbiturates (abc-BAR)
Maalox	Aluminium Hydroxide	No Cross reaction
Maalox TC	Aluminium Hydroxide	No Cross reaction
Magnapen	Co-Fluampicil	No Cross reaction
Magnapen	Flucloxacillin	No Cross reaction
Marevan	Warfarin Sodium	No Cross reaction
Marinol	Dronabinol (Delta-9-Tetrahydrocannabinol)	Positive on Cannabis (abc-THC))
Maxidex	Dexamethasone	No Cross reaction
Maxidrol	Dexamethasone	No Cross reaction
Medacepan	Medazepam	Positive on Benzodiazepine (abc-BZD)
Medinal	Barbital	Positive on Barbiturates (abc-BAR)
Medinex	Diphenhydramine	No Cross reaction
Mefoxin	Penicillin	No Cross reaction
Mellaril	Thioridazine Hydrochloride	No Cross reaction
Merbentyl	Dicyclomine	No Cross reaction
Methadose	Methadone Hydrochloride	Positive on Methadone (abc-MTD)
Methamprex	d-Methamphetamine HCL	Positive on Methamphetamine (abc-MET)
Methazine	Promethazine	Positive on Tricyclic Antidepressants (abc-TCA)
Methedrine	d-Methamphetamine HCL	Positive on Methamphetamine (abc-MET)
Metrogel	Metronidazole	No Cross reaction
Metrolyl	Metronidazole	No Cross reaction
Metrotop	Metronidazole	No Cross reaction
Micronor	Norethindrone	No Cross reaction
Migraleve	Codeine Phosphate	Positive on Opiates (abc-OPI)
Mildison lipocream	Hydrocortisone	No Cross reaction
Minims	Dexamethasone	No Cross reaction
Mogadon	Nitrazepam	Positive on Benzodiazepine (abc-BZO)
Molipaxin	Trazodone	No Cross reaction



Cross reactivities on abc rapid drug tests

Medicine (Trade name) Content / Active substance Result when tested with abc

Morcap	Morphine	Positive für Opiates (MOR/OPI)
Morphine Sulphate	Morphine	Positive on Opiates (abc-OPI)
Motifene	Diclofenac Sodium	No Cross reaction
Motrin	Ibuprofen	No Cross reaction
MS Contin	Morphine	Positive on Opiates (abc-OPI)
Mucogel	Aluminium Hydroxide	No Cross reaction
Myambutol	Ethambutol	No Cross reaction
Nalone	Naloxone	Positive on Opiates (abc-OPI)
Nalorex	Naltrexone	No Cross reaction
Napratec	Naproxen	No Cross reaction
Naprosen	Naproxen	No Cross reaction
Naprosyn	Naproxen	No Cross reaction
Narcan	Naloxone	Positive on Opiates (abc-OPI)
Narcotine	Noscapine	No Cross reaction
Naseptin	Chlorhexidine Gluconate	No Cross reaction
Nasobec	Beclometasone	No Cross reaction
Nemactil	Pericyazine	No Cross reaction
Nembutal	Pentobarbital	Positive on Barbiturates (abc-BAR)
Neo-cortef	Hydrocortisone	No Cross reaction
Neo-Naclax	Bendrofluazide	No Cross reaction
Neo-Naclax-K	Bendrofluazide	No Cross reaction
Neo-synephrine	Oxymetazoline	No Cross reaction
Neulactil	Pericyazine	No Cross reaction
Neuratop	Metronidazole	No Cross reaction
Neurocil	Levomepromazin	No Cross reaction
Nicoderm	Nicotine	No Cross reaction
Nicorette	Nicotine	No Cross reaction
Nicotinell	Nicotine	No Cross reaction
Nicotrol	Nicotine	No Cross reaction
Nifar	Nifedipine	No Cross reaction
Nifecard	Nifedipine	No Cross reaction
Nif-ten	Nifedipine	No Cross reaction
Niquitin	Nicotine	No Cross reaction
Nobritol	Medazepam	Positive on Benzodiazepine (abc-BZD)
Nobrium	Medazepam	Positive on Benzodiazepine (abc-BZD)
Noctamide	Lormetazepam	Positive on Benzodiazepine (abc-BZD)
Norflex	Orphenadrine	No Cross reaction
Norgesic	Orphenadrine	No Cross reaction
Noriday	Norethindrone	No Cross reaction
Norimide	Loperamide HCl	No Cross reaction
Norimin	Norethindrone	No Cross reaction
Norinyl	Norethindrone	No Cross reaction
Noristat	Norethindrone	No Cross reaction
Noritrate	Metronidazole	No Cross reaction
Norlutin	Norethindrone	No Cross reaction
Norsynephrine	d/l-Octopamine	No Cross reaction
Novocain	Procaine	Positive on Opiates (abc-OPI)
Nuvelle	Norethindrone	No Cross reaction



Cross reactivities on abc rapid drug tests

Medicine (Trade name) Content / Active substance Result when tested with abc

Nycopren	Naproxen	No Cross reaction
Nystaform	Chlorhexidine Gluconate	No Cross reaction
Nystaform	Hydrocortisone	No Cross reaction
Nytol	Diphenhydramine	No Cross reaction
Olynth	Xylometazolin	No Cross reaction
Opazimes	Koalin und Morphine Mixture	Positive on Opiates (abc-OPI)
Oramorph	Morphine	Positive on Opiates (abc-OPI)
Orfiril	Sodium Valproate	No Cross reaction
Orfiril	Valproinsäure	No Cross reaction
Orudis	Ketapofen	No Cross reaction
Oruvali	Ketapofen	No Cross reaction
Otomize	Dexamethasone	No Cross reaction
Otriven	Xylometazolin	No Cross reaction
Otrivine-Anistin	Antazoline Sulphate	No Cross reaction
Ovysmen	Norethindrone	No Cross reaction
Ox-pam	Oxazepam	Positive on Benzodiazepine (abc-BZD)
Panadol	Acetaminophen (Siehe auch Paracetamol)	No Cross reaction
Panadol Night	Diphenhydramine	No Cross reaction
Paradote	Paracetamol (Acetaminophen)	No Cross reaction
Paramol	Dihydrocodeine	Positive on Opiates (abc-OPI)
Pavacol-D	Pholcodine	Positive on Opiates (abc-OPI)
Paxil	Paroxetine	No Cross reaction
Pediapred	Prednisolone	No Cross reaction
Pediatric BP	Codeine Phosphate	Positive on Opiates (abc-OPI)
Penbritin	Ampicillin	No Cross reaction
Perazin-neuraxpharm	Perazin-Dimalonat	No Cross reaction
Perinal	Hydrocortisone	No Cross reaction
Phenacetin	Acetophenetidin	No Cross reaction
Phenergen	Promethazine	No Cross reaction
Physetone	Methadone Hydrochloride	Positive on Methadone (abc-MTD)
Pipracil	Piperacillin	No Cross reaction
Piriton	Chlorpheniramine Maleate	No Cross reaction
Plaquenil	Hydroxychloroquine	No Cross reaction
Polycillin	Ampicillin	No Cross reaction
Ponstan	Mefenamic Acid	No Cross reaction
Ponstel	Mefenamic Acid	No Cross reaction
Powergel	Ketoprofen	No Cross reaction
Prelone	Prednisolone	No Cross reaction
Prestim	Bendrofluazide	No Cross reaction
Principen	Ampicillin	No Cross reaction
Procardia	Nifedipine	No Cross reaction
Proctofoam	Hydrocortisone	No Cross reaction
Proflex	Ibuprofen	No Cross reaction
Propetyl	Pericyazine	No Cross reaction
ProSom	Estazolam	Positive on Benzodiazepine (abc-BZD)
Prothiaden	Dothiepin	No Cross reaction
Prozac	Fluoxetine	No Cross reaction
Psycholept	Pericyazine	No Cross reaction



Cross reactivities on abc rapid drug tests

Medicine (Trade name)

Content / Active substance

Result when tested with abc

Pylorid	Ranitidine	Positive on Methamphetamine (abc-MET)
Pyrogastrone	Aluminium Hydroxide	No Cross reaction
Quinocort	Hydrocortisone	No Cross reaction
Qvar	Beclometasone	No Cross reaction
Reglan	Metoclopramide	No Cross reaction
Remedeine	Dihydrocodeine	Positive on Opiates (abc-OPI)
Restandol	Testosterone	No Cross reaction
Restoril	Temazepam	Positive on Benzodiazepine (abc-BZD)
Ritalin	Methylphenidate	No Cross reaction
Rivotril	Clonazepam	Positive on Benzodiazepine (abc-BZD)
Roferon-A	Interferon Alfa	No Cross reaction
Rohypnol	Flunitrazepam	Positive on Benzodiazepine (abc-BZD)
Roxanol	Morphine	Positive on Opiates (abc-OPI)
Rozex	Metronidazole	No Cross reaction
Sarafem	Fluoxetine	No Cross reaction
Scopoderm	Hyoscine butylbromide	No Cross reaction
Seconal	Secobarbital	Positive für Barbiturates (BAR)
Securopen	Azlocillin	No Cross reaction
Senokot	Sennosides	No Cross reaction
Serax	Oxazepam	Positive on Benzodiazepine (abc-BZD)
Seroquel	Quetiapin	Positive in high concentrations on Tricyclic Antidepressants (abc-TCA)
Serotulle	chlorhexidine Gluconate	No Cross reaction
Seroxat	Paroxetine	No Cross reaction
Severedol	Morphine	Positive on Opiates (abc-OPI)
Sinequan	Doxepin	Positive on Tricyclic Antidepressants (abc-TCA)
slang name „anabolic steroids“	Testosterone	No Cross reaction
Sofradex	Dexamethasone	No Cross reaction
Solpadol	Codeine Phosphate, Paracetamol	Positive on Opiates (abc-OPI)
Somnite	Nitrazepam	Positive on Benzodiazepine (abc-BZD)
Soneryl	Butabarbital	Positive on Barbiturates (abc-BAR)
Sorbichew	Isosorbide Dinitrate	No Cross reaction
Sorbid SA	Isosorbide Dinitrate	No Cross reaction
Sorbitrate	Isosorbide Dinitrate	No Cross reaction
Spalt	Ibuprofen	No Cross reaction
Spasmonal	Alverine Citrate	No Cross reaction
Spasmonal Fibre	Alverine Citrate	No Cross reaction
Steripod	chlorhexidine Gluconate	No Cross reaction
Stesolid	Diazepam	Positive on Benzodiazepine (abc-BZD)
Stiemycin	Erythromycin	No Cross reaction
Stilnoct	Zolpidem	No Cross reaction
Strong BP	Pholcodine	Positive on Opiates (abc-OPI)
Substitol ret.	Retardiertes Morphine sulfate	Positive on Opiates (abc-OPI)
Subutex	Buprenorphine	Positive für Buprenorphin (abc-BUP)
Sudafed	Pseudoephedrine	No Cross reaction
Sufenta	Sufentanil	No Cross reaction
Sustanon	Testosterone	No Cross reaction



Cross reactivities on abc rapid drug tests

Medicine (Trade name)

Content / Active substance

Result when tested with abc

Sustiva	Efavirenz	Positive on Cannabis (abc-THC) * (Only the Metabolite from urinesamples, Not the substance itself)
Synflex	Naproxen	No Cross reaction
Tagamet	Cimetidine	No Cross reaction
Taxilan	Perazin-Dimalonat	No Cross reaction
Tazobactam	Piperacillin	No Cross reaction
Tegretal	Carbamazin	No Cross reaction
Tegretol	Carbamazepine	No Cross reaction
Temgesic	Buprenorphine	Positive on Buprenorphine (abc-BUP)
Tempra	Acetaminophen (siehe auch Paracetamol)	No Cross reaction
Tenben	Atenolol	No Cross reaction
Tenben	Bendrofluazide	No Cross reaction
Tenif	Atenolol	No Cross reaction
Tenorectic	Atenolol	No Cross reaction
Tenoret50	Atenolol	No Cross reaction
Tenormin	Atenolol	No Cross reaction
Teril	Carbamazepine	No Cross reaction
Terra-cortil	Hydrocortisone	No Cross reaction
Terra-Cortil	Oxytetracycline	No Cross reaction
Terramycin	Oxytetracycline	No Cross reaction
Thebacon	Pholcodine	Positive on Opiates (abc-OPI)
Tiloryth	Erythromycin	No Cross reaction
Timodine	Hydrocortisone	No Cross reaction
Timonil	Carbamazepine	No Cross reaction
Timonil	Carbamazin	No Cross reaction
Tisept	Chlorhexidine Gluconate	No Cross reaction
Topal	Aluminium Hydroxide	No Cross reaction
Topamax	Topiramat	No Cross reaction
Tramake	Tramadol	No Cross reaction
Tramal	Tramadol	No Cross reaction on Opiates and Buprenorphin
Tranxene	Chlorazepate	Positive on Benzodiazepine (abc-BZD)
Trepergen	Co-Phenotrope (atropine/diphenoxylate)	No Cross reaction
Trexan	Naltrexone	No Cross reaction
Trialodine	Trazodone	No Cross reaction
Trimovate	Oxytetracycline	No Cross reaction
TriNovum	Norethindrone	No Cross reaction
Triptafen	Amitriptyline	Positive on Tricyclic Antidepressants (abc-TCA)
Triptafen-M	Amitriptyline	Positive on Tricyclic Antidepressants (abc-TCA)
Trisquens	Norethindrone	No Cross reaction
Tryptizol	Amitriptyline	Positive on Tricyclic Antidepressants (abc-TCA)
Tuinal	Amobarbital	Positive on Barbiturates (abc-BAR)
Tylenol	Acetaminophen (siehe auch Paracetamol)	No Cross reaction
Tylenol 3	Acetaminophen with Codeine	Positive on Opiates (abc-OPI)
Tylenol 3	Paracetamol (Acetaminophen)/ Codeine Preparations	Positive on Opiates (abc-OPI)



Cross reactivities on abc rapid drug tests

Medicine (Trade name) Content / Active substance Result when tested with abc

Tylenol Cold (combination)	Pseudoephedrine	No Cross reaction
Tylenol with Codeine	Acetaminophen with Codeine	Positive on Opiates (abc-OPI)
Tylex	Codeine Phosphate	Positive on Opiates (abc-OPI)
Tylex	Paracetamol (Acetaminophen)/ Codeine Preparations	Positive on Opiates (abc-OPI)
Ucerax	Hydroxyzine Hydrochloride	No Cross reaction
Ultiva	Remifentanyl	No Cross reaction
Ultram	Tramadol	No Cross reaction
Uniroid-HC	Hydrocortisone	No Cross reaction
Unisept	Chlorhexidine Gluconate	No Cross reaction
Uriflex	Chlorhexidine Gluconate	No Cross reaction
Uro-Trainer	Chlorhexidine Gluconate	No Cross reaction
Utovian	Norethindrone	No Cross reaction
Utrogest	Progesteron	No Cross reaction
Valclair	Diazepam	Positive on Benzodiazepine (abc-BZD)
Valium	Diazepam	Positive on Benzodiazepine (abc-BZD)
Valoron N	Tilidin, in Kombination mit Naloxone	Positive on Opiates (abc-OPI)
Vendal ret.	Retardiertes Morphine hydrochloride	Positive on Opiates (abc-OPI)
Venotlin	Salbutamol	No Cross reaction
Ventide	Beclometasone	No Cross reaction
Ventide	Salbutamol	No Cross reaction
Ventodisks	Salbutamol	No Cross reaction
Veronal	Barbital	Positive on Barbiturates (abc-BAR)
Vibramycin	Doxycycline	No Cross reaction
Vibramycin-D	Doxycycline	No Cross reaction
Vick's Inhaler	l-Methamphetamine HCL	Positive on Methamphetamine (abc-MET)
Vicodin	Hydrocodone	Positive on Opiates (abc-OPI)
Vioform-HC	Hydrocortisone	No Cross reaction
Viraferon	Interferon Alfa	No Cross reaction
Viormone	Testosterone	No Cross reaction
Volmax	Salbutamol	No Cross reaction
Volraman	Diclofenac Sodium	No Cross reaction
Volsaid	Diclofenac Sodium	No Cross reaction
Voltarol	Diclofenac Sodium	No Cross reaction
Wellbutrin	Bupropion	No Cross reaction
Welldorm	Chloral Hydrate	No Cross reaction
Wellferon	Interferon Alfa	No Cross reaction
Wildnil	Carfentanyl	No Cross reaction
Xanax	Alprazolam	Positive on Benzodiazepine (abc-BZD)
Xepin	Doxepin	Positive on Tricyclic Antidepressants (abc-TCA)
Ximovan	Zopiclon	No Cross reaction
XTC (slang)	Methylenedioxyamphetamine (MDMA)	Positive on Methamphetamine (abc-MET) ans Ecstasy (MDMA)
Xyloproct	Hydrocortisone	No Cross reaction
Zamadol	Tramadol	No Cross reaction
Zantac	Ranitidine	Positive on Methamphetamine (abc-MET)
Zestoretic	Lisinopril HCl	No Cross reaction
Zestril	Lisinopril HCl	No Cross reaction



Cross reactivities on abc rapid drug tests

Medicine (Trade name)

Content / Active substance

Result when tested with abc

Zidoval	Methronidazole	No Cross reaction
Zineryl	Erythromycin	No Cross reaction
Zita	Cimetidine	No Cross reaction
Zoloft	Sertraline	No Cross reaction
Zonivent	Beclometasone	No Cross reaction
Zop Filmdoublette	Zopiclon	No Cross reaction
Zopiclon	Zopiclon	No Cross reaction
Zoton	Lansoprazole	No Cross reaction
Zyban	Bupropion	No Cross reaction
Zydol	Tramadol	No Cross reaction
Zymoet	Metronidazole	No Cross reaction
Zyprexa	Olanzapine	No Cross reaction

Revision: 8.02.2008 (BG)