



## Cross reactivities on abc rapid drug tests

Content / Active substance

Medicine (Trade name)

Result when tested with abc

Acetaminophen (see also Paracetamol)	Acephen	No Cross reaction
Acetaminophen (see also Paracetamol)	Aceta	No Cross reaction
Acetaminophen (see also Paracetamol)	Apacet	No Cross reaction
Acetaminophen (see also Paracetamol)	Dapacen	No Cross reaction
Acetaminophen (see also Paracetamol)	Excedrin (combination)	No Cross reaction
Acetaminophen (see also Paracetamol)	Feverall	No Cross reaction
Acetaminophen (see also Paracetamol)	Panadol	No Cross reaction
Acetaminophen (see also Paracetamol)	Tempra	No Cross reaction
Acetaminophen (see also Paracetamol)	Tylenol	No Cross reaction
Acetaminophen with Codeine	Tylenol 3	Positive on Opiates (abc-OPI)
Acetaminophen with Codeine	Tylenol with Codeine	Positive on Opiates (abc-OPI)
Acetophenetidin	Phenacetin	No Cross reaction
Acetylsalicylic acid, Aspirin, ASS	Anadin	No Cross reaction
Acetylsalicylic acid, Aspirin, ASS	Anasin	No Cross reaction
Acetylsalicylic acid, Aspirin, ASS	Angettes	No Cross reaction
Acetylsalicylic acid, Aspirin, ASS	ASA	No Cross reaction
Acetylsalicylic acid, Aspirin, ASS	Asasantin	No Cross reaction
Acetylsalicylic acid, Aspirin, ASS	Aspirin	No Cross reaction
Acetylsalicylic acid, Aspirin, ASS	Bayer Aspirin	No Cross reaction
Acetylsalicylic acid, Aspirin, ASS	Bufferin	No Cross reaction
Acetylsalicylic acid, Aspirin, ASS	Caprin	No Cross reaction
Acetylsalicylic acid, Aspirin, ASS	Caprin	No Cross reaction
Acetylsalicylic acid, Aspirin, ASS	Disprin	No Cross reaction
Acetylsalicylic acid, Aspirin, ASS	Ecotrin	No Cross reaction
Acetylsalicylic acid, Aspirin, ASS	Empirin	No Cross reaction
Acetylsalicylic acid, Aspirin, ASS	Excedrin	No Cross reaction
Acetylsalicylic acid, Aspirin, ASS	Excedrin (combination)	No Cross reaction
Alfentanil	Alfenta	No Cross reaction
Allobarbital	Curral	Positive on Barbiturates (abc-BAR)
Alprazolam	Xanax	Positive on Benzodiazepines (abc-BZD)
Aluminium Chloride Hexahydrate	Anhydrol Forte	No Cross reaction
Aluminium Chloride Hexahydrate	Drichlor	No Cross reaction
Aluminium Hydroxide	Gastrocote	No Cross reaction
Aluminium Hydroxide	Kolanticon	No Cross reaction
Aluminium Hydroxide	Maalox	No Cross reaction
Aluminium Hydroxide	Maalox TC	No Cross reaction
Aluminium Hydroxide	Mucogel	No Cross reaction
Aluminium Hydroxide	Pyrogastrone	No Cross reaction
Aluminium Hydroxide	Topal	No Cross reaction
Aluminium Hydroxyde	Alisone	No Cross reaction
Aluminium Hydroxyde	Alu-Cap	No Cross reaction
Alverine Citrate	Spasmonal	No Cross reaction
Alverine Citrate	Spasmonal Fibre	No Cross reaction
Amitriptyline	Elavil	Positive on Tricyclic Antidepressants (abc-TCA)
Amitriptyline	Lentizole	Positive on Tricyclic Antidepressants
Amitriptyline	Triptafen	Positive on Tricyclic Antidepressants
Amitriptyline	Triptafen-M	Positive on Tricyclic Antidepressants



## Cross reactivities on abc rapid drug tests

Content / Active substance

Medicine (Trade name)

Result when tested with abc

Amitriptyline	Tryptizol	Positive on Tricyclic Antidepressants
Ammonia/Ipecacuanha	Ipecac	No Cross reaction
Amobarbital	Amytal	Positive on Barbiturates (abc-BAR)
Amobarbital	Tuinal	Positive on Barbiturates (abc-BAR)
Amoxicillin oder Amoxycillin	Amoram	No Cross reaction
Amoxicillin oder Amoxycillin	Amoxil	No Cross reaction
Amoxicillin oder Amoxycillin	Augmentin	No Cross reaction
Amoxicillin oder Amoxycillin	Heliclear	No Cross reaction
Ampicillin	Penbritin	No Cross reaction
Ampicillin	Polycillin	No Cross reaction
Ampicillin	Principen	No Cross reaction
Antazoline Sulphate	Otrivine-Anistin	No Cross reaction
Aprobarbital	Alurate	Positive on Barbiturates (abc-BAR)
Atenolol	Beta-adlat	No Cross reaction
Atenolol	Co-tenidone	No Cross reaction
Atenolol	Kalten	No Cross reaction
Atenolol	Tenben	No Cross reaction
Atenolol	Tenif	No Cross reaction
Atenolol	Tenorectic	No Cross reaction
Atenolol	Tenoret50	No Cross reaction
Atenolol	Tenormin	No Cross reaction
Atosil	Promethazin	Positive on Opiates (abc-OPI)
Atorvastatin	Lipitor	No Cross reaction
Azlocillin	Securopen	No Cross reaction
Barbital	Medinal	Positive on Barbiturates (abc-BAR)
Barbital	Veronal	Positive on Barbiturates (abc-BAR)
Beclometasone	AeroBec	No Cross reaction
Beclometasone	AeroBec Forte	No Cross reaction
Beclometasone	Asmabec	No Cross reaction
Beclometasone	Beclazone	No Cross reaction
Beclometasone	Becloforte	No Cross reaction
Beclometasone	Becodisks	No Cross reaction
Beclometasone	Beconase	No Cross reaction
Beclometasone	Becotide	No Cross reaction
Beclometasone	Fillair	No Cross reaction
Beclometasone	Nasobec	No Cross reaction
Beclometasone	Qvar	No Cross reaction
Beclometasone	Ventide	No Cross reaction
Beclometasone	Zonivent	No Cross reaction
Bendrofluazide	Aprinox	No Cross reaction
Bendrofluazide	Cogaretic	No Cross reaction
Bendrofluazide	Inderex	No Cross reaction
Bendrofluazide	Indertic	No Cross reaction
Bendrofluazide	Neo-Naclax	No Cross reaction
Bendrofluazide	Neo-Naclax-K	No Cross reaction
Bendrofluazide	Prestim	No Cross reaction
Bendrofluazide	Tenben	No Cross reaction
Benperidol (Butyrophenone)	Glianimon	No Cross reaction



## Cross reactivities on abc rapid drug tests

Content / Active substance

Medicine (Trade name)

Result when tested with abc

Benzalkonium	Bradosol	No Cross reaction
Benzydamine HCl	Difflam	No Cross reaction
Benzylpenicillin	Crystapen	No Cross reaction
Biperiden	Akineton	No Cross reaction
Bromazepam	Lexotan	Positive on Benzodiazepine (abc-BZD)
Bromopheniramine	Dimetapp	No Cross reaction
Bromopheniramine	Dimotane	No Cross reaction
Bromopheniramine	Dimotapp	No Cross reaction
Buprenorphine	Subutex	Positive on Buprenorphine (abc-BUP)
Buprenorphine	Temgesic	Positive on Buprenorphine (abc-BUP)
Bupropion	Wellbutrin	No Cross reaction
Bupropion	Zyban	No Cross reaction
Butabarbital	Butisol	Positive on Barbiturates (abc-BAR)
Butabarbital	Butisol	Positive on Barbiturates (abc-BAR)
Butabarbital	Soneryl	Positive on Barbiturates (abc-BAR)
Butalbital	Fioricet	Positive on Barbiturates (abc-BAR)
Butalbital	Fiorinal	Positive on Barbiturates (abc-BAR)
Butobarbital	Butethal	Positive on Barbiturates (abc-BAR)
Canestan	Canestan ears/skin/vaginitis/HC	No Cross reaction
Carbamazepine	Tegretol	No Cross reaction
Carbamazepine	Teril	No Cross reaction
Carbamazepine	Timonil	No Cross reaction
Carbamazin	Finlepsin	No Cross reaction
Carbamazin	Tegretal	No Cross reaction
Carbamazin	Timonil	No Cross reaction
Carfentanyl	Wildnil	No Cross reaction
Cephalexin	Ceporex	No Cross reaction
Cephalexin	Keflex	No Cross reaction
Chloral Hydrate	Welldorm	No Cross reaction
Chlorazepate	Tranxene	Positive on Benzodiazepine (abc-BZD)
Chlordiazepoxide	Librium	Positive on Benzodiazepine (abc-BZD)
Chlorhexidine Gluconate	Instillagel	No Cross reaction
Chlorhexidine gluconate	Bactrigras	No Cross reaction
Chlorhexidine Gluconate	Cordsodyl	No Cross reaction
Chlorhexidine Gluconate	CX Powder	No Cross reaction
Chlorhexidine Gluconate	Dermol	No Cross reaction
Chlorhexidine Gluconate	Hibicet	No Cross reaction
Chlorhexidine Gluconate	Hibiscrub	No Cross reaction
Chlorhexidine Gluconate	Hibisol	No Cross reaction
Chlorhexidine Gluconate	Hibitane	No Cross reaction
Chlorhexidine Gluconate	Naseptin	No Cross reaction
Chlorhexidine Gluconate	Nystaform	No Cross reaction
chlorhexidine Gluconate	Serotulle	No Cross reaction
chlorhexidine Gluconate	Steripod	No Cross reaction
Chlorhexidine Gluconate	Tisept	No Cross reaction
Chlorhexidine Gluconate	Unisept	No Cross reaction
Chlorhexidine Gluconate	Uriflex	No Cross reaction
Chlorhexidine Gluconate	Uro-Trainer	No Cross reaction



## Cross reactivities on abc rapid drug tests

Content / Active substance

Medicine (Trade name)

Result when tested with abc

Chlorhexidines Gluconate	Chlorohex	No Cross reaction
Chlorpheniramine Maleate	Galpseud	No Cross reaction
Chlorpheniramine Maleate	Haymine	No Cross reaction
Chlorpheniramine Maleate	Piriton	No Cross reaction
Chlorpromazine	Largactil	No Cross reaction
Cimetidine	Dyspamet	No Cross reaction
Cimetidine	Tagamet	No Cross reaction
Cimetidine	Zita	No Cross reaction
Citalopram	Cipramil	No Cross reaction
Clobazam	Frisium	Positive on Benzodiazepine (abc-BZD)
Clonazepam	Clonopin	Positive on Benzodiazepine (abc-BZD)
Clonazepam	Klonopin	Positive on Benzodiazepine (abc-BZD)
Clonazepam	Rivotril	Positive on Benzodiazepine (abc-BZD)
Co-Amoxyclav	Augmentin	No Cross reaction
Co-Amoxyclav	Augmentin Duo	No Cross reaction
Codeine Phosphate	Codafen Continus	Positive on Opiates (abc-OPI)
Codeine Phosphate	Codeine Linctus	Positive on Opiates (abc-OPI)
Codeine Phosphate	Galcodine	Positive on Opiates (abc-OPI)
Codeine Phosphate	Kapake	Positive on Opiates (abc-OPI)
Codeine Phosphate	Migraleve	Positive on Opiates (abc-OPI)
Codeine Phosphate	Pediatric BP	Positive on Opiates (abc-OPI)
Codeine Phosphate	Tylex	Positive on Opiates (abc-OPI)
Codeine Phosphate, Paracetamol	Solpadol	Positive on Opiates (abc-OPI)
Co-Fluampicil	Magnapen	No Cross reaction
Co-Phenotrope (atropine/diphenoxylate)	Lomotil	No Cross reaction
Co-Phenotrope (atropine/diphenoxylate)	Trepergen	No Cross reaction
d/l-Octopamine	Norsynephrine	No Cross reaction
Delorazepam	Briantum	Positive on Benzodiazepine (abc-BZD)
Dexamethasone	Decadron	No Cross reaction
Dexamethasone	Dexa-Rhinaspray Duo	No Cross reaction
Dexamethasone	Maxidex	No Cross reaction
Dexamethasone	Maxidrol	No Cross reaction
Dexamethasone	Minims	No Cross reaction
Dexamethasone	Otomize	No Cross reaction
Dexamethasone	Sofradex	No Cross reaction
Dexamphetamine Sulphate	Adderall	Positive on Amphetamines (abc-AMP)
Dexamphetamine Sulphate	Adderall XR	Positive on Amphetamines (abc-AMP)
Dexamphetamine Sulphate	Dexedrine	Positive on Amphetamines (abc-AMP)
Dextropropoxyphene	Co-proximal	Positive on Propoxyphene (abc-PPX)
Dextropropoxyphene	Darvocet	Positive on Propoxyphene (abc-PPX)
Dextropropoxyphene	Darvon	Positive on Propoxyphene (abc-PPX)
Diazepam	Diazemuls	Positive on Benzodiazepine (abc-BZD)
Diazepam	Stesolid	Positive on Benzodiazepine (abc-BZD)
Diazepam	Valclair	Positive on Benzodiazepine (abc-BZD)
Diazepam	Valium	Positive on Benzodiazepine (abc-BZD)
Diclofenac Sodium	Dicloflex	No Cross reaction
Diclofenac Sodium	Diclomax	No Cross reaction
Diclofenac Sodium	Motifene	No Cross reaction



# Cross reactivities on abc rapid drug tests

Content / Active substance

Medicine (Trade name)

Result when tested with abc

Diclofenac Sodium	Volraman	No Cross reaction
Diclofenac Sodium	Volsaid	No Cross reaction
Diclofenac Sodium	Voltarol	No Cross reaction
Dicyclomine	Kolanticon	No Cross reaction
Dicyclomine	Merbentyl	No Cross reaction
Dihydrocodeine	DHC Continus	Positive on Opiates (abc-OPI)
Dihydrocodeine	Paramol	Positive on Opiates (abc-OPI)
Dihydrocodeine	Remedeine	Positive on Opiates (abc-OPI)
Dimenthicone	Fancorsil	No Cross reaction
Dimethicone	Asilone	No Cross reaction
Diphenhydramine	Benadryl	No Cross reaction
Diphenhydramine	Medinex	No Cross reaction
Diphenhydramine	Nytol	No Cross reaction
Diphenhydramine	Panadol Night	No Cross reaction
d-Methamphetamine HCL	Desoxyn	Positive on Methamphetamine (abc-MET)
d-Methamphetamine HCL	Methamprex	Positive on Methamphetamine (abc-MET)
d-Methamphetamine HCL	Methedrine	Positive on Methamphetamine (abc-MET)
Dothiepin	Prothiaden	No Cross reaction
Doxepin	Sinequan	Positive on Tricyclic Antidepressants (abc-TCA)
Doxepin	Xepin	Positive on Tricyclic Antidepressants (abc-TCA)
Doxycycline	Vibramycin	No Cross reaction
Doxycycline	Vibramycin-D	No Cross reaction
Dronabinol (Delta-9-Tetrahydrocannabinol)	Marinol	Positive on Cannabis (abc-THC)
Droperidol	Dropletan	No Cross reaction
Efavirenz	Sustiva	Positive on Cannabis (abc-THC) * (Only the Metabolite from urinesamples, Not the substance itself)
Erythromycin	Arpimycin	No Cross reaction
Erythromycin	Benzymycin	No Cross reaction
Erythromycin	Erycane	No Cross reaction
Erythromycin	Erymax	No Cross reaction
Erythromycin	Erythrocin	No Cross reaction
Erythromycin	Erythroped	No Cross reaction
Erythromycin	Illosone	No Cross reaction
Erythromycin	Isotrexin	No Cross reaction
Erythromycin	Lactobinate	No Cross reaction
Erythromycin	Stiemycin	No Cross reaction
Erythromycin	Tiloryth	No Cross reaction
Erythromycin	Zineryl	No Cross reaction
Estazolam	ProSom	Positive on Benzodiazepine (abc-BZD)
Ethambutol	Myambutol	No Cross reaction
Ethylmorphine	Dionin	Positive on Opiates (abc-OPI)
Fentanyl	Actiq	No Cross reaction
Flucloxacillin	Co-fluampicil	No Cross reaction
Flucloxacillin	Floxapen	No Cross reaction
Flucloxacillin	Magnapen	No Cross reaction
Flunitrazepam	Rohypnol	Positive on Benzodiazepine (abc-BZD)



## Cross reactivities on abc rapid drug tests

Content / Active substance

Medicine (Trade name)

Result when tested with abc

Fluoxetine	Prozac	No Cross reaction
Fluoxetine	Sarafem	No Cross reaction
Flupentixol decanoate	Depixol	No Cross reaction
Flupentixol decanoate	Fluanxol	No Cross reaction
Fusidic Acid (Sodium Fusidate)	Fucibet	No Cross reaction
Fusidic Acid (Sodium Fusidate)	Fucidin	No Cross reaction
Fusidic Acid (Sodium Fusidate)	Fucithalmic	No Cross reaction
Gentamicin Sulphate	Cidomycin	No Cross reaction
Gentamicin Sulphate	Genticin	No Cross reaction
Gentamicin Sulphate	Gentisone	No Cross reaction
Haloperidol	Haldol	No Cross reaction
Hydrocodone	Loratab	Positive on Opiates (abc-OPI)
Hydrocodone	Lorcet	Positive on Opiates (abc-OPI)
Hydrocodone	Vicodin	Positive on Opiates (abc-OPI)
Hydrocortisone	Actinac	No Cross reaction
Hydrocortisone	Alphaderm	No Cross reaction
Hydrocortisone	Alphosyl-HC	No Cross reaction
Hydrocortisone	Anugesic-HC	No Cross reaction
Hydrocortisone	Anusol-HC	No Cross reaction
Hydrocortisone	Calmurid-HC	No Cross reaction
Hydrocortisone	Canesten-HC	No Cross reaction
Hydrocortisone	Colioam	No Cross reaction
Hydrocortisone	Daktacort	No Cross reaction
Hydrocortisone	Dioderm	No Cross reaction
Hydrocortisone	Econacort	No Cross reaction
Hydrocortisone	Efcortesel	No Cross reaction
Hydrocortisone	Efortelan	No Cross reaction
Hydrocortisone	Eurax-HC	No Cross reaction
Hydrocortisone	Gregoderm	No Cross reaction
Hydrocortisone	Hydrocortistab	No Cross reaction
Hydrocortisone	Hydrocortistab	No Cross reaction
Hydrocortisone	Hydrocortone	No Cross reaction
Hydrocortisone	Hydrocortone	No Cross reaction
Hydrocortisone	Mildison lipocream	No Cross reaction
Hydrocortisone	Neo-cortef	No Cross reaction
Hydrocortisone	Nystaform	No Cross reaction
Hydrocortisone	Perinal	No Cross reaction
Hydrocortisone	Proctofoam	No Cross reaction
Hydrocortisone	Quinocort	No Cross reaction
Hydrocortisone	Terra-cortil	No Cross reaction
Hydrocortisone	Timodine	No Cross reaction
Hydrocortisone	Uniroid-HC	No Cross reaction
Hydrocortisone	Vioform-HC	No Cross reaction
Hydrocortisone	Xyloproct	No Cross reaction
Hydrocortisone-17-butyrate	Locoid	No Cross reaction
Hydrocortisone-17-butyrate	Locoid C	No Cross reaction
Hydromorphone	Dilaudid	Positive on Opiates (abc-OPI)
Hydromorphone	Hydrostat	Positive on Opiates (abc-OPI)



## Cross reactivities on abc rapid drug tests

Content / Active substance

Medicine (Trade name)

Result when tested with abc

Hydromorphone	Hydrostat	Positive on Opiates (abc-OPI)
Hydroxychloroquine	Plaquenil	No Cross reaction
Hydroxyzine Hydrochloride	Atarax	No Cross reaction
Hydroxyzine Hydrochloride	Ucerax	No Cross reaction
Hyoscine butylbromide	Buscopan	No Cross reaction
Hyoscine butylbromide	Scopoderm	No Cross reaction
Ibuprofen	Brufen	No Cross reaction
Ibuprofen	Codafen	No Cross reaction
Ibuprofen	Dolormin	No Cross reaction
Ibuprofen	Fenbid	No Cross reaction
Ibuprofen	Ibugel	No Cross reaction
Ibuprofen	Ibuspray	No Cross reaction
Ibuprofen	Motrin	No Cross reaction
Ibuprofen	Proflex	No Cross reaction
Ibuprofen	Spalt	No Cross reaction
l-Methamphetamine HCL	Vick's Inhaler	Positive on Methamphetamines (abc-MET)
Indomethacin	Flexin continus	No Cross reaction
Indomethacin	Indocid	No Cross reaction
Indomethacin	Indomod	No Cross reaction
Interferon Alfa	Intron A	No Cross reaction
Interferon Alfa	Roferon-A	No Cross reaction
Interferon Alfa	Viraferon	No Cross reaction
Interferon Alfa	Wellferon	No Cross reaction
Isoniazide	INH	No Cross reaction
Isosorbide Dinitrate	Angitak	No Cross reaction
Isosorbide Dinitrate	Cedocard	No Cross reaction
Isosorbide Dinitrate	Isocard	No Cross reaction
Isosorbide Dinitrate	Isoket	No Cross reaction
Isosorbide Dinitrate	Isordil	No Cross reaction
Isosorbide Dinitrate	Sorbichew	No Cross reaction
Isosorbide Dinitrate	Sorbid SA	No Cross reaction
Isosorbide Dinitrate	Sorbitrate	No Cross reaction
Ketaprofen	Orudis	No Cross reaction
Ketaprofen	Oruvali	No Cross reaction
Ketaprofen	Powergel	No Cross reaction
Koalin und Morphin Mixture	Entersan	Positive on Opiates (abc-OPI)
Koalin und Morphine Mixture	Diocalm	Positive on Opiates (abc-OPI)
Koalin und Morphine Mixture	Opazimes	Positive on Opiates (abc-OPI)
Lactulose	Duphalec	No Cross reaction
Lactulose	Lactugal	No Cross reaction
Lamotrigine	Lamitcal	No Cross reaction
Lansoprazole	Heliclear	No Cross reaction
Lansoprazole	Zoton	No Cross reaction
Levomepromazin	Levium	No Cross reaction
Levomepromazin	Neurocil	No Cross reaction
Lisinopril HCl	Carace	No Cross reaction
Lisinopril HCl	Zestoretic	No Cross reaction
Lisinopril HCl	Zestril	No Cross reaction



# Cross reactivities on abc rapid drug tests

Content / Active substance

Medicine (Trade name)

Result when tested with abc

Lofepramine	Gamanil	Positive on Tricyclic Antidepressants
Lofepramine	Lomont	Positive on Tricyclic Antidepressants
Lofexidine	Britoflex	No Cross reaction
Loperamide HCl	Imodium	No Cross reaction
Loperamide HCl	Loperagen	No Cross reaction
Loperamide HCl	Norimide	No Cross reaction
Loratadine	Claritin	No Cross reaction
Lorazepam	Ativan	Positive on Benzodiazepine (abc-BZD)
Lormetazepam	Noctamide	Positive on Benzodiazepine (abc-BZD)
Lysergic acid diethylamide	LSD	No Cross reaction
Magnesium Alginate	Gaviscon	No Cross reaction
Mannitol	Gaviscon	No Cross reaction
Medazepam	Anxitol	Positive on Benzodiazepine (abc-BZD)
Medazepam	Lerisum	Positive on Benzodiazepine (abc-BZD)
Medazepam	Medacepan	Positive on Benzodiazepine (abc-BZD)
Medazepam	Nobritol	Positive on Benzodiazepine (abc-BZD)
Medazepam	Nobrium	Positive on Benzodiazepine (abc-BZD)
Mefenamic Acid	Ponstan	No Cross reaction
Mefenamic Acid	Ponstel	No Cross reaction
Meperidine	Demerol	No Cross reaction
Methadone Hydrochloride	Dolophine	Positive on Methadone (abc-MTD)
Methadone Hydrochloride	Methadose	Positive on Methadone (abc-MTD)
Methadone Hydrochloride	Physetone	Positive on Methadone (abc-MTD)
Methronidazole	Elyzol	No Cross reaction
Methronidazole	Zidoval	No Cross reaction
Methylenedioxyamphetamine (MDA)	Eve (slang)	Positive on Amphetamines (abc-AMP) und Ecstasy (MDMA)
Methylenedioxyamphetamine (MDA)	Love Drug (slang)	Positive on Amphetamines (abc-AMP) and Ecstasy (abc-MDMA)
Methylenedioxymethamphetamine (MDMA)	Adam (slang)	Positive on Methamphetamines (abc-MET) and Ecstasy (abc-MDMA)
Methylenedioxymethamphetamine (MDMA)	E (slang)	Positive on Methamphetamines (abc-MET) and Ecstasy (abc-MDMA)
Methylenedioxymethamphetamine (MDMA)	Ecstasy	Positive on Methamphetamines (abc-MET) and Ecstasy (abc-MDMA)
Methylenedioxymethamphetamine (MDMA)	XTC (slang)	Positive on Methamphetamines (abc-MET) and Ecstasy (abc-MDMA)
Methylphenidate	Ritalin	No Cross reaction
Metoclopramide	Reglan	No Cross reaction
Metronidazole	Anabact	No Cross reaction
Metronidazole	Flagyl	No Cross reaction
Metronidazole	Metrogel	No Cross reaction
Metronidazole	Metrolyl	No Cross reaction
Metronidazole	Metrotop	No Cross reaction
Metronidazole	Neuratop	No Cross reaction
Metronidazole	Noritrate	No Cross reaction
Metronidazole	Rozex	No Cross reaction
Metronidazole	Zymoet	No Cross reaction
Morphine	Astramorph	Positive on Opiates (abc-OPI)
Morphine	Cyclimorph	Positive on Opiates (abc-OPI)





## Cross reactivities on abc rapid drug tests

Content / Active substance

Medicine (Trade name)

Result when tested with abc

Morphine	Duramorph	Positive on Opiates (abc-OPI)
Morphine	Morcap	Positive on Opiates (abc-OPI)
Morphine	Morphine Sulphate	Positive on Opiates (abc-OPI)
Morphine	MS Contin	Positive on Opiates (abc-OPI)
Morphine	Oramorph	Positive on Opiates (abc-OPI)
Morphine	Roxanol	Positive on Opiates (abc-OPI)
Morphine	Severedol	Positive on Opiates (abc-OPI)
Nadolol	Corgard	No Cross reaction
Nadolol	Corgaretic	No Cross reaction
Nadolol	Corgartic	No Cross reaction
Naloxone	Nalone	Positive on Opiates (abc-OPI)
Naloxone	Narcan	Positive on Opiates (abc-OPI)
Naltrexone	Antaxone	No Cross reaction
Naltrexone	Nalorex	No Cross reaction
Naltrexone	Trexan	No Cross reaction
Naproxen	Aleve	No Cross reaction
Naproxen	Condrotec	No Cross reaction
Naproxen	Naprtec	No Cross reaction
Naproxen	Naprosen	No Cross reaction
Naproxen	Naprosyn	No Cross reaction
Naproxen	Nycopren	No Cross reaction
Naproxen	Synflex	No Cross reaction
Nicotine	Nicoderm	No Cross reaction
Nicotine	Nicorette	No Cross reaction
Nicotine	Nicotinell	No Cross reaction
Nicotine	Nicotrol	No Cross reaction
Nicotine	Niquitin	No Cross reaction
Nifedipine	Adalat	No Cross reaction
Nifedipine	Nifar	No Cross reaction
Nifedipine	Nifecard	No Cross reaction
Nifedipine	Nif-ten	No Cross reaction
Nifedipine	Procardia	No Cross reaction
Nitrazepam	Mogadon	Positive on Benzodiazepine (abc-BZD)
Nitrazepam	Somnite	Positive on Benzodiazepine (abc-BZD)
Norethindrone	BiNovum	No Cross reaction
Norethindrone	Brevinor	No Cross reaction
Norethindrone	Climagest	No Cross reaction
Norethindrone	Climesse	No Cross reaction
Norethindrone	Elleste Duet	No Cross reaction
Norethindrone	Estracombi	No Cross reaction
Norethindrone	Evorel	No Cross reaction
Norethindrone	Kliofem	No Cross reaction
Norethindrone	Kliovance	No Cross reaction
Norethindrone	Loestrin	No Cross reaction
Norethindrone	Micronor	No Cross reaction
Norethindrone	Noriday	No Cross reaction
Norethindrone	Norimin	No Cross reaction
Norethindrone	Norinyl	No Cross reaction



## Cross reactivities on abc rapid drug tests

Content / Active substance

Medicine (Trade name)

Result when tested with abc

Norethindrone	Noristat	No Cross reaction
Norethindrone	Norlutin	No Cross reaction
Norethindrone	Nuvelle	No Cross reaction
Norethindrone	Ovysmen	No Cross reaction
Norethindrone	TriNovum	No Cross reaction
Norethindrone	Trisquens	No Cross reaction
Norethindrone	Utovian	No Cross reaction
Noscapine	Narcotine	No Cross reaction
Olanzapine	Zyprexa	No Cross reaction
Orphenadrine	Disipil	No Cross reaction
Orphenadrine	Norflex	No Cross reaction
Orphenadrine	Norgesic	No Cross reaction
Oxazepam	Ox-pam	Positive on Benzodiazepine (abc-BZD)
Oxazepam	Serax	Positive on Benzodiazepine (abc-BZD)
Oxymetazoline	Afrin	No Cross reaction
Oxymetazoline	Neo-syneprhine	No Cross reaction
Oxytetracycline	Terra-Cortil	No Cross reaction
Oxytetracycline	Terramycin	No Cross reaction
Oxytetracycline	Trimovate	No Cross reaction
Paracetamol (Acetaminophen)	Fortagesic	No Cross reaction
Paracetamol (Acetaminophen)	Paradote	No Cross reaction
Paracetamol (Acetaminophen) /Codeine Preparations	Kapake	Positive on Opiates (abc-OPI)
Paracetamol (Acetaminophen)/ Codeine Preparations	Tylenol 3	Positive on Opiates (abc-OPI)
Paracetamol (Acetaminophen)/ Codeine Preparations	Tylex	Positive on Opiates (abc-OPI)
Paracetamol (Acetaminophen)/ Codeine Preparations	Co-codamol	Positive on Opiates (abc-OPI)
Paracetamol (Acetaminophen)/ Codeine Preparations	Codafen	Positive on Opiates (abc-OPI)
Paracetamol (Acetaminophen)/ Codeine Preparations	Co-dydramol	Positive on Opiates (abc-OPI)
Paroxetine	Paxil	No Cross reaction
Paroxetine	Seroxat	No Cross reaction
Penicillin	Combicillin	No Cross reaction
Penicillin	Mefoxin	No Cross reaction
Pentobarbital	Nembutal	Positive on Barbiturates (abc-BAR)
Perazin-Dimalonat	Perazin-neuraxpharm	No Cross reaction
Perazin-Dimalonat	Taxilan	No Cross reaction
Pericyazine	Amplan	No Cross reaction
Pericyazine	Aolept	No Cross reaction
Pericyazine	Apamin	No Cross reaction
Pericyazine	Iryakin	No Cross reaction
Pericyazine	Nemactil	No Cross reaction
Pericyazine	Neulactil	No Cross reaction
Pericyazine	Propetyl	No Cross reaction
Pericyazine	Psycholept	No Cross reaction
Pethidine (meperidine)	Demerol	No Cross reaction
Pethidine (meperidine)	Dolantin	No Cross reaction



# Cross reactivities on abc rapid drug tests

Content / Active substance

Medicine (Trade name)

Result when tested with abc

Phenobarbital	Donnatal	Positive on Barbiturates (abc-BAR)
Phenobarbitone (sehen auch Phenobarbital)	Luminal	Positive on Barbiturates (abc-BAR)
Phenytoin	Dilantin	No Cross reaction
Phenytoin	Epanutin	No Cross reaction
Phenytoin	Epitard	No Cross reaction
Pholcodine	Galenphol	Positive on Opiates (abc-OPI)
Pholcodine	Pavacol-D	Positive on Opiates (abc-OPI)
Pholcodine	Strong BP	Positive on Opiates (abc-OPI)
Pholcodine	Thebacon	Positive on Opiates (abc-OPI)
Piperacillin	Pipracil	No Cross reaction
Piperacillin	Tazobactam	No Cross reaction
Prazepam	Centrax	Positive on Benzodiazepine (abc-BZD)
Prazepam	Demetrin	Positive on Benzodiazepine (abc-BZD)
Prednisolone	Delta-cortef	No Cross reaction
Prednisolone	Econopred	No Cross reaction
Prednisolone	Inflamase	No Cross reaction
Prednisolone	Pediapred	No Cross reaction
Prednisolone	Prelone	No Cross reaction
Procaine	Novocain	Positive on Opiates (abc-OPI)
Progesteron	Utrogest	No Cross reaction
Promethazine	Allerfen	Positive on Tricyclic Antidepressants (abc-TCA)
Promethazine	Anergan	Positive on Tricyclic Antidepressants (abc-TCA)
Promethazine	Antinaus	Positive on Tricyclic Antidepressants (abc-TCA)
Promethazine	Aprobit	Positive on Tricyclic Antidepressants (abc-TCA)
Promethazine	Avomine	Positive on Tricyclic Antidepressants (abc-TCA)
Promethazine	Baymethazine	Positive on Tricyclic Antidepressants (abc-TCA)
Promethazine	Diprozin	No Cross reaction
Promethazine	Fenergan	Positive on Tricyclic Antidepressants (abc-TCA)
Promethazine	Methazine	Positive on Tricyclic Antidepressants (abc-TCA)
Promethazine	Phenergen	No Cross reaction
Propranolol	Beta-Progane	No Cross reaction
Propranolol	Inderal	No Cross reaction
Propranolol	Inderetic	No Cross reaction
Propranolol	Inderex	No Cross reaction
Pseudoephedrine	Afrinol	No Cross reaction
Pseudoephedrine	Sudafed	No Cross reaction
Pseudoephedrine	Tylenol Cold (combination)	No Cross reaction
Quetiapin	Seroquel	Positive in high concentrations on Tricyclic Antidepressants (abc-TCA)
Ranitidine	Pylorid	Positive on Tricyclic Antidepressants (abc-TCA)
Ranitidine	Zantac	Positive on Methamphetamine (abc-MET)
Remifentanil	Ultiva	No Cross reaction
Retardiertes Dihydrocodein	Codidol ret.	Positive on Opiates (abc-OPI)



# Cross reactivities on abc rapid drug tests

Content / Active substance

Medicine (Trade name)

Result when tested with abc

Retardiertes Morphine hydrochloride	Compensan ret.	Positive on Opiates (abc-OPI)
Retardiertes Morphine hydrochloride	Vendal ret.	Positive on Opiates (abc-OPI)
Retardiertes Morphine sulfate	Kapanol ret.	Positive on Opiates (abc-OPI)
Retardiertes Morphine sulfate	Substitol ret.	Positive on Opiates (abc-OPI)
Salbutamol	Aerocrom	No Cross reaction
Salbutamol	Aerolin	No Cross reaction
Salbutamol	Airomir	No Cross reaction
Salbutamol	Asmasal	No Cross reaction
Salbutamol	Combivent	No Cross reaction
Salbutamol	Duovent	No Cross reaction
Salbutamol	Ventolin	No Cross reaction
Salbutamol	Ventide	No Cross reaction
Salbutamol	Ventodisks	No Cross reaction
Salbutamol	Volmax	No Cross reaction
Secobarbital	Seconal	Positive on Barbiturates (abc-BAR)
Sennosides	Senokot	No Cross reaction
Sertraline	Zoloft	No Cross reaction
Sodium Valproate	Convulex	No Cross reaction
Sodium Valproate	Depakene	No Cross reaction
Sodium Valproate	Depakote	No Cross reaction
Sodium Valproate	Epilim	No Cross reaction
Sodium Valproate	Ergenyl	No Cross reaction
Sodium Valproate	Orfiril	No Cross reaction
Sufentanil	Sufenta	No Cross reaction
Temazepam	Restoril	Positive on Benzodiazepine (abc-BZD)
Testosterone	Andropatch	No Cross reaction
Testosterone	Restandol	No Cross reaction
Testosterone	slang name „anabolic steroids“	No Cross reaction
Testosterone	Sustanon	No Cross reaction
Testosterone	Viormone	No Cross reaction
Thioridazine Hydrochloride	Mellaril	No Cross reaction
Thyroxin Sodium	Eltroxin	No Cross reaction
Tilidin, in Kombination mit Naloxone	Valoron N	Positive on Opiates (abc-OPI)
Topiramate	Topamax	No Cross reaction
Tramadol	Tramake	No Cross reaction
Tramadol	Tramal	No Cross reaction on Opiates (abc-OPI) and Buprenorphin (abc-BUP)
Tramadol	Ultram	No Cross reaction
Tramadol	Zamadol	No Cross reaction
Tramadol	Zydol	No Cross reaction
Trazodone	Desyrel	No Cross reaction
Trazodone	Molipaxin	No Cross reaction
Trazodone	Trialodine	No Cross reaction
Triazolam	Halcion	Positive on Benzodiazepine (abc-BZD)
Valproinsäure	Orfiril	No Cross reaction
Venlafaxine	Effexor	Potentially Positive on PCP (abc-PCP)
Venlafaxine	Effexor XL	Potentially Positive on PCP (abc-PCP)
Warfarin Sodium	Coumadin	No Cross reaction



## Cross reactivities on abc rapid drug tests

Content / Active substance

Medicine (Trade name)

Result when tested with abc

Warfarin Sodium	Marevan	No Cross reaction
Xylometazolin	Olynth	No Cross reaction
Xylometazolin	Otriven	No Cross reaction
Zolpidem	Stilnoct	No Cross reaction
Zopiclon	Ximovan	No Cross reaction
Zopiclon	Zop Filmtablette	No Cross reaction
Zopiclon	Zopiclon	No Cross reaction

Revision: 8.02.2008 (BG)